

SCHEDULE FOR HYGIENE AND MAINTENANCE

The following steps are to be taken to ensure hygiene of the internal fluid path and external surfaces of the machine, in order to maintain maximum performance of all dialysis machines.

Frequency	Measures	Cleaning	Decalcification	Disinfection
After each treatment	<ol style="list-style-type: none"> 1. heat disinfect 2. Clean surface with neutral detergent and 70% ethanol for VRE patients 			x
Daily	CleanCart-C cartridge		x	x
Weekly	CleanCart - A This should be performed directly after a Heat disinfection with Cleancart C following afternoon treatment.	x	x	x
Monthly AK200 Ultra	Change water filter and dialysis fluid filter followed by a heat disinfect.			x
When time after disinfection exceeds 16 hours but is less than seven days AK 200 Ultra	Heat disinfect before treatment			x
When time after disinfection has exceeded seven days AK 200 Ultra	<ol style="list-style-type: none"> 1) Change water filter and dialysis fluid filter. 2) Heat disinfection program before treatment 			x